



Ivory® LinguaGuard™

Safe and effective retraction—made simple.

A disposable, convenient add-on that allows for retraction and protection of the tongue during suction, by attaching to the vented end of a high volume evacuator (HVE) tip.



Giving a hand to oral health.



KULZER
MITSUI CHEMICALS GROUP

“Patient tongues are remarkably good at escaping from restraint and finding their way into a danger zone. LinguaGuard completely eliminates that worry! With gentle pressure, I’m able to hold the tongue back and suction with one hand, freeing up my right hand for passing instruments, handling materials, or rinsing.”

- Shannon Pace Brinker, CDA CDD



Features

- Simple design slides easily onto a high volume evacuator (HVE) tip*
- Wrap design ensures strong fit and support
- Curved design ensures comfortable retraction
- Larger surface area makes the HVE tip safer and more effective
- Easily retract without risk of tissue suck
- Meant for single use and easily disposable
- BPA and Latex free

*HVE tip sold separately



Instructions for Use

- Spin the HVE tip as you insert it into the LinguaGuard sleeve. Spin until the vent hole aligns with the protruding tab in the LinguaGuard sleeve, locking it in place.
- The retractor should be curved away from the tip of the HVE tip.
- To remove, spin LinguaGuard counterclockwise to disengage it from the HVE tip vent hole.

Disposal

- LinguaGuard should not be sterilized or reused. It should be properly discarded in a clinical waste container to prevent transmission of infectious pathogens.

Ivory® LinguaGuard™

66076088

LinguaGuard - 100 count, HVE retraction aids

Also from Kulzer:

Ivory® ReLeaf™

Dentistry’s best hands-free suction solution.

- Hands-free, high-volume efficiency
- Soft, comfortable and universal mouthpiece for your patients
- Simple setup—connects to existing HVE system in < 60 seconds



View From A Patient

“I found ReLeaf to be very comfortable...I also liked that I had control over the water and fluid buildup as I was able to close and remove all fluid whenever I wanted to.”